# S40

**CABINA AUDIOMÉTRICA  
AUDIOMETRIC TEST BOOTH**

**MANUAL DEL USUARIO  
USER'S MANUAL**

501-700-MUM · Rev. 2.02 · 2021-04

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Audiometric test booth **S40**  
Revision: 501-700-MUM Rev 2.02  
All rights reserved

This manual can be purchased through the After Sales Service.

**SIBEL S.A.U.**

Rosellón 500 bajos, 08026 BARCELONA (Spain)  
National Sales: Tel. +34 93 436 00 08 e-mail: [comercial@sibelmed.com](mailto:comercial@sibelmed.com)  
International Sales: Tel. +34 93 436 00 07 e-mail: [export@sibelmed.com](mailto:export@sibelmed.com)  
Technical Service: Tel. +34 93 433 54 50 e-mail: [sat@sibelmed.com](mailto:sat@sibelmed.com)  
Fax: +34 93 436 16 11, Web: [www.sibelmed.com](http://www.sibelmed.com)

**ADVICE ABOUT THE COPYRIGHT**

No part of this publication may be reproduced, transmitted, transcribed, stored in a back-up system or translated into any language or computer language in any form or by any means, electronic, mechanical, optical, chemical or manual without the express written consent from SIBEL S.A.U.

**DISCLAIMER**

**SIBEL S.A.U.** is responsible for the security, reliability and performance of this product only if:

- The place where the system is installed meets the requirements of electric installation IEC and other applicable regulations.
- Repairs, revisions or modifications, within and outside the warranty period are carried out by technical personnel of SIBEL S.A.U.
- The product is used by qualified staff in accordance with the recommendations of this User's Manual

**PRODUCT IN COMPLIANCE WITH MEDICAL  
DEVICE REGULATION (EU) 2017/745  
(Class I).**

Revised  
Date: 2021-04  
Technical Director

Approved  
Date: 2021-04  
Commercial Director

**INDEX**

<b>1. INSTALLATION AND OPERATING INSTRUCTIONS .....</b>	<b>4</b>
1.1. INTRODUCTION .....	4
1.2. PRELIMINARY OBSERVATIONS .....	4
1.3. DISPOSAL OF WASTE ELECTRICAL AND ELECTRONIC EQUIPMENT BY USERS IN PRIVATE HOUSEHOLDS IN THE EUROPEAN UNION .....	5
<b>2. MAIN FEATURES .....</b>	<b>5</b>
2.1. PACKING LIST .....	6
2.2. OVERVIEW OF THE BOOTH .....	7
2.3. INSTALLATION AND COMMISSIONING .....	8
2.3.1. INSTALLATION .....	8
2.3.2. COMMISSIONING .....	8
<b>3. SYMBOLS .....</b>	<b>9</b>
<b>4. TECHNICAL SPECIFICATIONS .....</b>	<b>10</b>
<b>5. STANDARDS .....</b>	<b>12</b>
<b>6. OPERATING PRINCIPLES .....</b>	<b>13</b>
<b>7. TREATMENT, PREVENTATIVE AND CORRECTIVE MAINTENANCE... 13</b>	
7.1. TREATMENT.....	13
7.2. PREVENTATIVE MAINTENANCE .....	14
7.2.1. MONITORING.....	14
7.2.2. INSPECTING THE INTERCONNECTIONS .....	14
7.3. CORRECTIVE MAINTENANCE.....	14

## 1. INSTALLATION AND OPERATING INSTRUCTIONS

### 1.1. INTRODUCTION

The **SIBELMED S40** soundproof booths have been designed by the **SIBEL S.A.U. R+D+i** department using the most up to date materials in acoustic installation.

The **SIBELMED S40** soundproof booth is an individual chamber that attenuates the sound waves between its exterior and interior. This sound attenuation feature allows audiometric tests to be performed in the noise environmental conditions specified by the standards related to how these types of measures should be carried out. (For further information, consult standards ANSI S3.1 and ISO 8253-1).

The booth has an access door with a lock, a window allowing the patient and operator to see each other, pleasant lighting and the necessary connections to connect an audiometer with all its accessories.

### 1.2. PRELIMINARY OBSERVATIONS

This booth has been manufactured following strict quality controls. However, accidents may occur during transportation or storage of the booths. For this reason, it is advisable to check the state of the booth before installing it, as well as the accessories that complement it.

#### **WARNING**

**IF THE PACKAGING IS DAMAGED IN ANY WAY, IMMEDIATELY CONTACT THE TRANSPORT AGENCY AND THEIR DISTRIBUTOR BEFORE PROCEEDING TO INSTALL. DO NOT DISCARD THE PACKAGING, BAGS, ETC., UNTIL CORRECT OPERATION OF THE EQUIPMENT HAS BEEN COMPLETELY VERIFIED.**

#### **WARNING**

**DO NOT STORE THE BOOTH UNASSEMBLED. DOING SO MAY CAUSE THE PANELS TO BECOME DEFORMED AND WILL VOID THE GUARANTEE.**

#### **WARNING**

**REPORT ANY SERIOUS INCIDENT TO THE MANUFACTURER AND COMPETENT AUTHORITY.**

### 1.3. DISPOSAL OF WASTE ELECTRICAL AND ELECTRONIC EQUIPMENT BY USERS IN PRIVATE HOUSEHOLDS IN THE EUROPEAN UNION

■ This symbol on the product indicates that the electrical part of this product may not be disposed of together with domestic waste.

If this type of waste must be disposed of, it is the responsibility of the user to disassemble the panel and the electrical part of the booth, and take it to a designated recycling collection point for electronic and electrical equipment. The recycling and separate collection of this waste at the time of disposal, will help to preserve natural resources; recycling protects health and the environment.

If you require additional information on places where this waste should be left for recycling, contact the domestic waste management department at your local authority or contact the distributor where you purchased the product.

## 2. MAIN FEATURES

The **S40** Soundproof Booth series is made up of the following models:

**S40-A** (90x90x213 cm)

**S40-B** (110x110x213 cm)

**S40-C** (127x127x213 cm)

**S40-D** (86x73x187 cm) (Mobile units)

**S40-E** (133 x 133 x 213 cm)

**S40-F** (Tailor-made projects)

## 2.1. PACKING LIST

The S40 soundproof booth comprises the following units and accessories:

<b>Code</b>	<b>Quantity</b>	<b>Description</b>
----	1	SIBELMED S40 BOOTH Model A,B,C,D,E or F
01217	1	MAINS POWER CABLE WITH EUROPEAN* PLUG
501-700-MU_	1	USER MANUAL
501-708-GR_	1	ASSEMBLY GUIDE
02214	5	AUDIOMETER-BOOTH INTERCONNECTION (6.3 mm)
----	1	AUDIOMETER-BOOTH INTERCONNECTION (3.5 mm)
09079, 09117	1	PANEL WITH 8 CONNECTIONS (OPC.) (Only models B, C, E, F)
02789	1	WHEEL SET MODELS A, D (OPC.)
08898	1	WHEEL SET MODELS B, C, E (OPC.)
03658	1	USB CABLE TYPE A-B 2.0

\* Inquire about other options

These codes may be used to order replacement parts for these options or accessories.

## 2.2. OVERVIEW OF THE BOOTH

**No. 1:** Window panel

**No. 2:** Window

**No. 3:** Connections board

**No. 4:** Folding table

**No. 5:** Table lock

**No. 6:** Edge pieces

**No. 7:** Roof

**No. 8:** Ventilation grill

**No. 9:** Rear panel


**No. 10:** Side panel

**No. 11:** Door frame

**No. 12:** Door

**No. 13:** Door handle

**No. 14:** Base

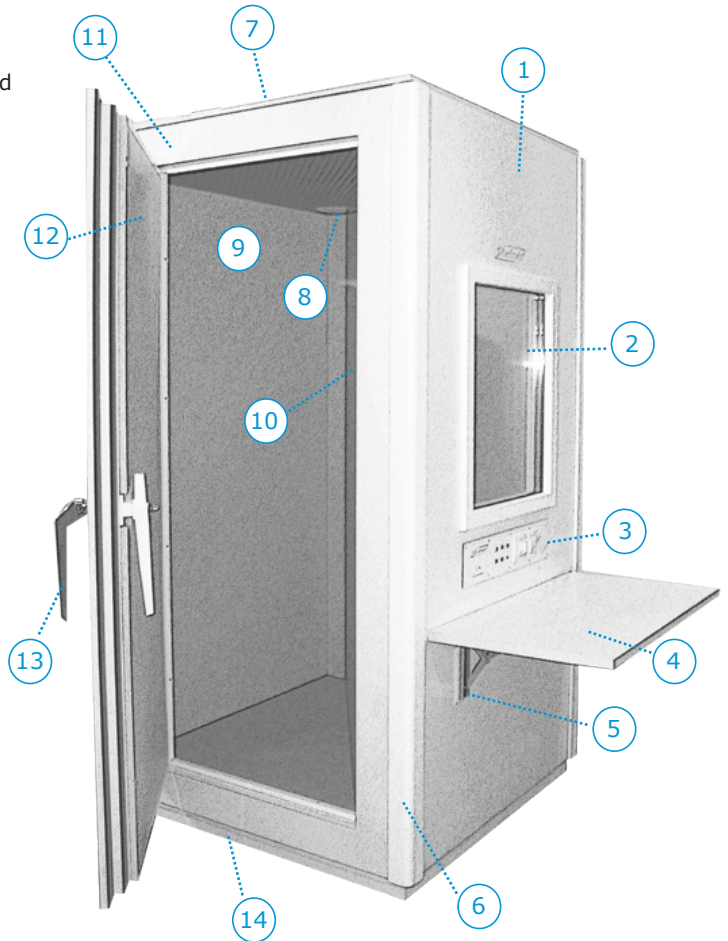## 2.3. INSTALLATION AND COMMISSIONING

### 2.3.1 INSTALLATION

Follow these steps to assemble the booth:

Choose the location where the booth is to be placed and its layout. Decide where the front panel is to be located, which includes the window and the table, and on what side of the front panel the door will be hung, as well as its opening direction.

Note: After selecting the location of the front panel, the rear panel will go on the opposite side; after choosing the location of the door, the side panel should go on the opposite side.

Check the assembly guide (501-708-GR\_) supplied with the booth for correct installation.

### 2.3.2. COMMISSIONING

Follow the steps below to start up the booth:

**NOTE:** It is presupposed that the user has an audiometer available to connect to the booth.

- Connect the mains power cable to the mains power supply input at the base of the booth, and then connect it to the mains power supply.
- Check that turning on the main switch (Nº3) lights up the interior of the booth. (If this is not the case, check the previous step and the mains power supply). Leave it switched off.
- Place the audiometer on the tray and connect its mains power supply cable to the output mains base (No. 3). **IMPORTANT NOTE: Maximum load 500 W.**
- Connect the audiometer accessories to the internal connections of the booth, and hang them on the supports available.
- Remember the connection number for each accessory to later connect the audiometer to the external connections of the booth (No.3), so that each accessory is correctly connected to the audiometer.

**WARNING**

**IF DURING THE INSTALLATION PROCESS OR IN ANY OTHER MOMENT THERE IS ANY DOUBT ON HOW TO PROCEED, CONTACT THE SIBEL S.A.U. AFTER-SALES SERVICE OR THE DISTRIBUTOR.**

**3. SYMBOLS**

MANUFACTURER (manufacture date, name and address of the manufacturer)

CATALOGUE NUMBER

SERIAL NUMBER

MEDICAL DEVICE

CE MARKING

CAUTION, ADDITIONAL WARNINGS IN INSTRUCTIONS FOR USE

REFER TO USER MANUAL

WARNING, RISK IDENTIFIED

WASTE OF ELECTRICAL AND ELECTRONIC EQUIPMENT

TEMPERATURE LIMITS

#### 4. TECHNICAL SPECIFICATIONS

**Description:** Individual soundproof booth for carrying out audiometric tests.

S40 RANGE MEASUREMENTS (cm.)		
CODE	MODELS	EXTERNAL
01634	S40-A	90 x 90 x 213
01636	S40-B	110 x 110 x 213
01637	S40-C	127 x 127 x 213
01639	S40-D	86 x 73 x 187
01640	S40-E	133 x 133 x 213
07107	S40-F	Special dimensions

**Window dimensions:** 70 x 58 cm. (May vary according to model)

#### Door dimensions:

Model S40-A	184.8 x 58.8 cm.
Model S40-B	184.8 x 78.8 cm.
Model S40-C	184.8 x 95.8 cm.
Model S40-D	162.7 x 58.8 cm.
Model S40-E	184.8 x 95.8 cm.

#### Folding table dimensions:

S40-A: 70 X 50 cm.

S40-B, C, E: 90 X 50 cm.

S40-D: 55 x 43 cm..

**Power supply:** 220 V - 50/60 Hz. (Other voltages are optional).  
Mains electrical circuit with switch and indicator light.

Internal lighting by LED lamp

Interior electrical consumption: less than 40 W.

Maximum load power: 500 W.

Booth passing connections: Five 6.3 mm Stereo female jack-type connections and one 3.5 mm female jack-type connection for connecting audiometer accessories.


Lifetime: 10 years

Global Attenuation for band white noise between 250 and 8000 Hz: 36 dB.

Maximum Attenuation for white noise: 49.7 dB at 5000 Hz

### **Dp insulation index by frequencies (model S40-B):**

f (Hz)	100	125	160	200	250	315	400
Dp (dB)	10.5	24.8	28.5	29.5	29.8	28.7	30.5
f (Hz)	500	630	800	1000	1250	1600	2000
Dp (dB)	32.4	34.2	36.1	37.9	37.1	39.1	39.8
f (Hz)	2500	3150	4000	5000	6300	8000	10000
Dp (dB)	43.3	45.9	47.5	49.7	44.1	45.0	45.6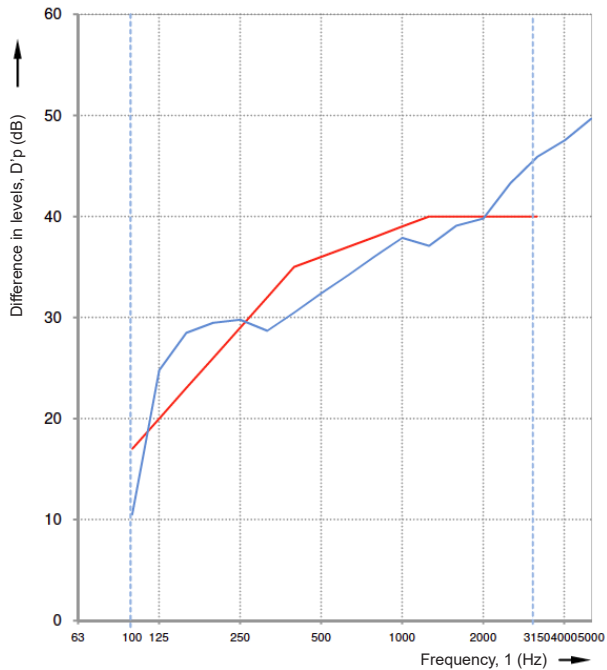

## 5. STANDARDS

**Audiometry:** Allows compliance with the ISO 8253-1 and ANSI S3.1 standards depending on the surrounding conditions.

**Quality:** Sibel Quality Systems (EN ISO 9001:2008 and EN ISO 13485:2016+AC:2018)

**Safety:** EN 60601-1:2006+AC:2010+A1:2013+AC:2014, EN 60601-1-6:2010+A1:2015, EN 62366:2008+A1:2015

**Risk management:** EN ISO 14971:2012

**Information supplied by the manufacturer:** EN 1041:2008+A1:2013

**Symbols used:** EN-ISO 15223-1:2016

## **6. OPERATING PRINCIPLES**

According to audiometry standards ANSI S3.1 and ISO 8253-1, audiometric tests should be carried out in conditions where environmental noise does not mask the results of these tests.

To this end, the booth is an acoustically-isolated enclosure where interior background noise is very low compared to outside noise, allowing audiometric tests to be performed correctly.

The booth has panels that contain different layers of sound-attenuating material. The basic principle that produces the sound attenuation is known as the law of mass. Other considerations to take into account in order to have good sound attenuation are based on the suitable combination of materials, as well as their dimensions, all of which avoids undesirable effects such, as the matching effect. (For further information, consult soundproofing and Acoustic conventions).

## **7. TREATMENT, PREVENTATIVE AND CORRECTIVE MAINTENANCE**

Like any other piece of equipment, the S40 soundproof booth requires treatment and maintenance, to guarantee the reliability and accuracy of the functions for which it has been designed. This implies carrying out a series of routine tasks.

### **7.1. TREATMENT**

The treatment is the action of maintaining the equipment in good working order; the person performing this task does not require any specific technical training, except knowledge of how to handle the equipment and its functions. Normally, this will be performed by the user of the equipment. The operations to be performed are:

The outside of the booth should be cleaned with soap and water, and then dried. On the inside of the booth, the foam walls and carpeted floor may be cleaned with a conventional vacuum cleaner: the plastic parts and the window may be cleaned with soap and water.

Do not use abrasive or solvent substances that may damage the finish of the booth.

## **7.2. PREVENTATIVE MAINTENANCE**

Preventative maintenance consists of all those actions that keep the equipment in good working order.

### **7.2.1. MONITORING**

The user should periodically monitor the condition of the booth, the internal lighting, the door lock and the state of the interconnections. Check that there are no broken elements or external damage.

### **7.2.2. INSPECTING THE INTERCONNECTIONS**

Biannually, check that the interconnections function correctly using an audiometer, and verify that they do not produce sounds that may reduce the reliability of the audiometric tests.

## **7.3. CORRECTIVE MAINTENANCE**

The corrective maintenance consists of leaving the equipment in good working order, when poor operation or improper use may have decreased its functionality or may have required repairs.

If a malfunction that impedes normal operation of the equipment is detected, contact Sibel, S.A.U. after-sales service, specifying, as detailed as possible, the nature of the anomaly.