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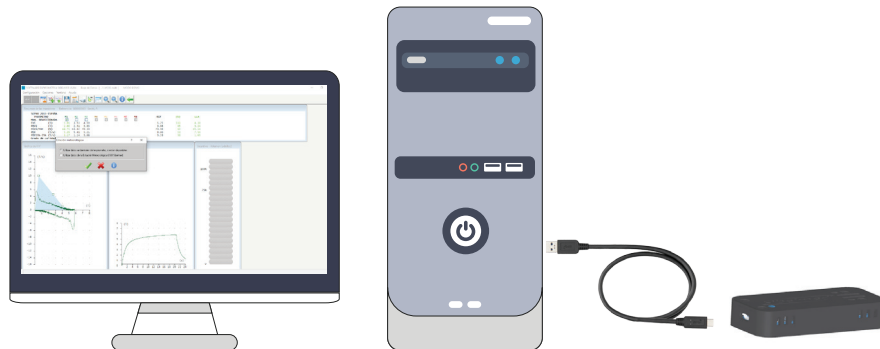
EN ESTATION METEOROLOGIQUE USB SIBELMED

COPYRIGHT

No part of this publication may be reproduced, transmitted, transcribed, stored in a back-up system or translated into any language or computer language in any form or by any means, electronic, mechanical, optical, chemical or manual without the express written consent from **SIBEL S.A.U.**

INTENDED USE

The USB weather station allows the acquisition of environmental temperature, pressure and humidity data and send them via USB. It is indicated for being used in conjunction with a Windows based PC and can be used with the W20s spirometry software and compatible **SIBELMED** spirometers.

INSTRUCTIONS FOR USE

Connect the weather station to a USB port of the PC via the supplied cable. No additional power supply is needed. The blue LED indicates that the weather station is running.

It is not necessary to install any driver on the PC.

It is recommended to keep the weather station connected to the PC for 30 min before starting measurements with the W20s software.

Weather data will be acquired automatically each time you perform a test, calibration or calibration check. (See scheme on the back).

WARNINGS

Do not place the weather station next to sources of heat and / or humidity.

Do not expose the system to direct sunlight.

Do not cover the holes in the housing.

Do not disconnect the cable from the station by pulling it. You can damage the station, reducing product safety. Always hold the cable by the connector.

Do not use the USB cable if it is damaged, since there is a risk of cramping.

Do not use the weather station outdoors or in moving transport vehicles.

Keep the weather station protected from shock and vibration. It may be damaged or malfunction.

Do not try to open the weather station. In case of failure or unexpected operation, please contact the **SIBEL S.A.U.** After Sales Service or your dealer.

DISPOSAL

Do not dispose the weather station and / or accessories with household waste. Deliver it in a designated collection point for recycling in accordance with the legal requirements of your country.

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